

Visby Sexual Health Test Information



go.visbymedical.com/training

Test Time	28 minutes
Test Storage	Temperature: 36°F – 86°F (2°C – 30°C) Humidity: 5% – 80% DO NOT freeze
Operating Conditions	Temperature: 55°F – 91°F (13°C – 33°C) Humidity: 5% – 80% Atmospheric Pressure: 300ft – 9500ft
Sample Type	Self Collected Vaginal Swab
Patient Sample Storage	4 hours at refrigerated to room temperature 36°F – 86°F (2°C – 30°C) 90 days at frozen temperature less than 5°F (-15°C)
Visby Sexual Health Test 10 Pack	• 10 Visby Medical Sexual Health Devices • 12 Pipettes • 1 Quick Reference Guide • Visby Medical Sexual Health Test Instructions for Use (available digitally)
Materials Required and Available as Accessories	• Visby Power Adapter • Visby Vaginal Specimen Collection Kit
Vaginal Specimen Collection Kit	• 50 Swabs • 50 Visby Collection Media Tubes • 50 Self-Collection Instructions • Vaginal Specimen Collection Kit Instructions for Use (available digitally)
Required but Not Supplied	• Medical Gloves • External Controls
External Controls	NATrol™ Chlamydia trachomatis (CT), Neisseria gonorrhoeae (NG), Trichomonas vaginalis (TV) Controls by ZeptoMetrix Corporation • Positive Control Packs: Catalog # NATCTNGTV-POS-IVD • Negative Control Packs: Catalog # NATCTNGTV-NEG-IVD
Need Help?	For Training: visby.training@visby.com For Customer Support: support@visby.com Call us: 1-833-468-4729 (1-833-GoVisby)